


## user manual

- **Medical advantages**  
safety in care
- **more comfort**  
for your patients
- **saving of labour**  
for your personnel
- **cost-reduction**  
for your institution

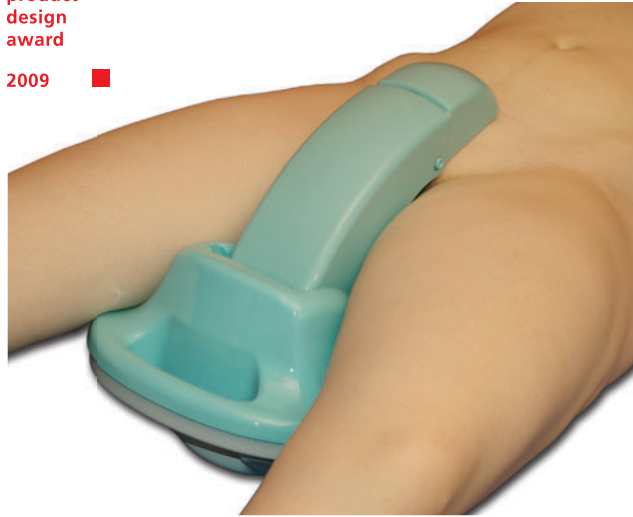

Dear customer,

You made an excellent decision by investing in the Cleanius bedpan which eases the situation of a bed-ridden person in a substantial way, and decreases nurse's workload significantly.

When we developed Cleanius Compact, we mainly focussed on ergonomics, hygiene as well as secure and easy handling. Another central topic was the protection of the intimate sphere.

Cleanius Compact and Cleanius Eco are medical products class I, certified to german guideline 93/42/EWG Anhang VII

### The difference

Cleanius Compact and Cleanius Eco differ in material and type of cleaning!



## Cleanius Compact

machine cleaning up to 110°C




## Cleanius Eco

manual cleaning up to 65°C

This user manual consists of:

- this information
- a Video-CD with movies guiding in handling

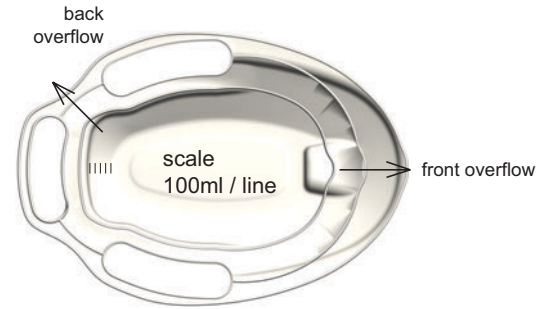

### cleaning and disinfection:

#### Cleanius Compact

will be cleaned and disinfected by a bedpan washer disinfector automatically.

#### Cleanius Eco

can be cleaned and disinfected with ordinary skin and plastic secure cleaners and disinfectants.

#### savety

Don't expose Cleanius to UV-light (sunlight) for a longer timeperiod. Don't use Cleanius under 12°C.

technical specifications can be requested from ServoClean.

#### appropriate application

The provided use of Cleanius Eco is the collection of human excrements and urine for people got bedridden. The liquid absorber is for gelatinizing urine only, don't eat, keep away from children.

#### extent of delivery

- 1x Cleanius
- 1x instruction manual
- 1x Video CD handling and information for Cleanius Eco additionally:
- 1x liquid absorber set (starter-Set)

#### accessories

- plastic liner and liquid absorber set:
- 50 set order # 51050
- 100 set order # 51100

#### sales

ServoClean UHS GbR.  
Weichselsteinerstraße 24  
D - 92369 Sengenthal  
Germany  
fon: 09181-29580  
fax: 09181-29588

info@cleanius.com  
www.cleanius.com

Use with a plastic line:



Put the plastic liner on the bottom part of Cleanius. Push in the plastic liner in the intake area,



and add the liquid absorber. Cleanius is ready to use.



Afterwards take out the plastic liner with gelized excrements dispose like a nappy.

2 Jahre warranty;  
original reception


manufacturer:  
Holzammer Kunststofftechnik  
Am Ursprung 9  
D- 92369 Sengenthal  
Germany

## Simple to handle.

Cleanius can be used regardless of the patients sex and allows collecting both, stool and urine at the same time. Apply Cleanius when the patient is in dorsal position, the waist of the patient may be up to an 45° "incline".


For lateral application the patient can be turned from or towards the nurse. Cleanius can be used dorsally as well.

### dorsal application

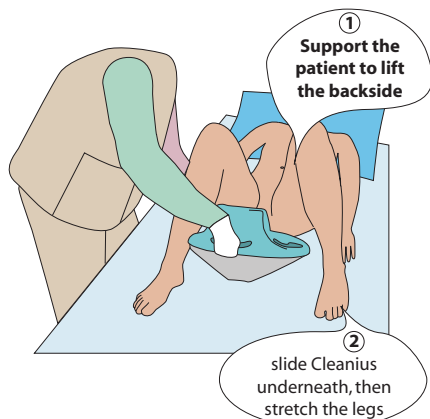


Positioning the patient correctly on Cleanius is achieved, when the human sacrum is lying right above the sacrum painted on the Cleanius bedpan.





This is the correct position.

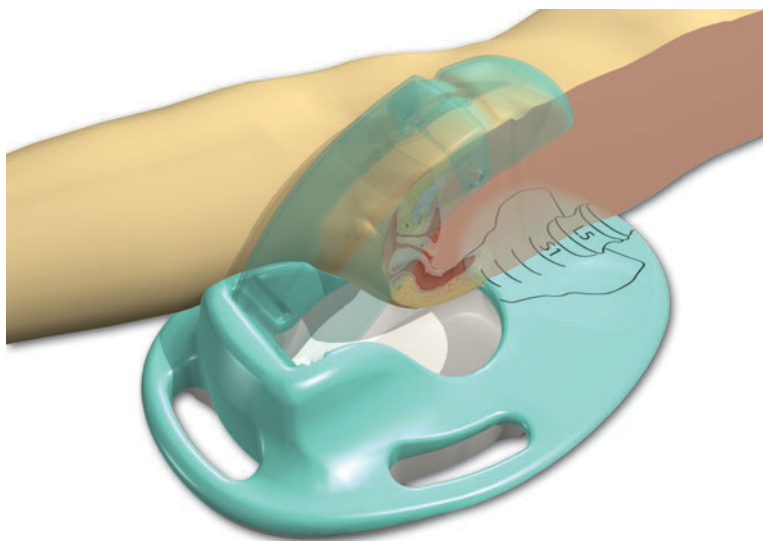

### lateral application

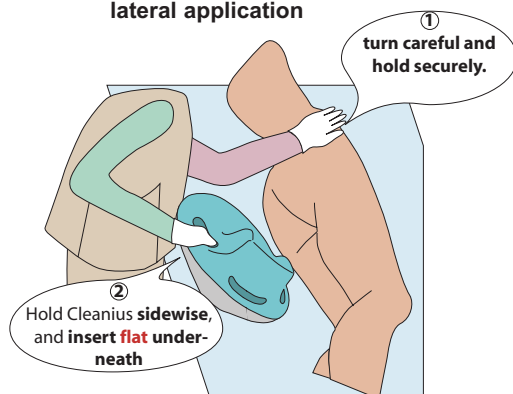

OR

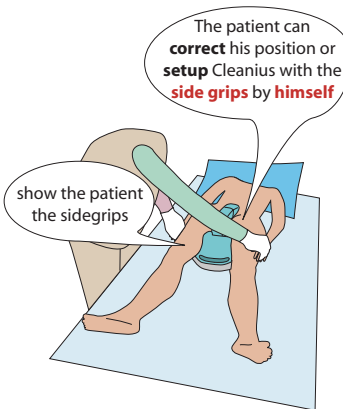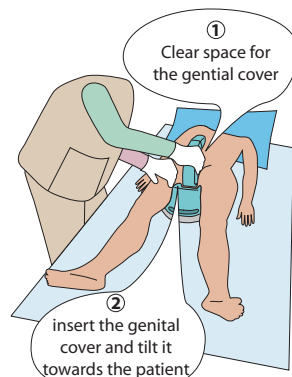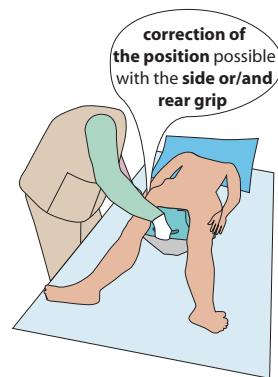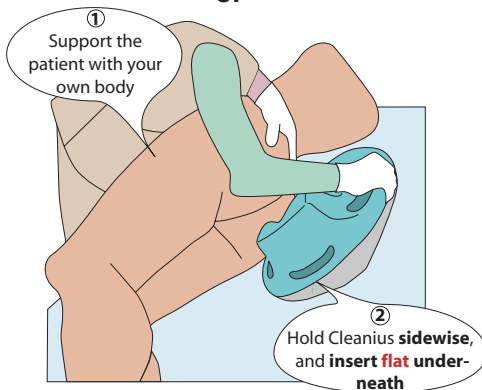

The following instructions describe the handling procedure for one nurse alone to properly place the patient on Cleanius. The patient must be able to lift his backside independently or the patient can be rolled to the side. Further situations may require a second nurse.

## handling-procedure

### Cleanius Compact and Cleanius Eco

#### before use

- bring Cleanius to the patient
- remove genitalcover
- remove the lid
- place the patient on Cleanius
- place the genitalcover \*

#### after use

- remove the genital cover
- remove Cleanius from the patient
- clean the patients buttocks
- insert the lid
- insert the genital cover

#### cleaning Cleanius Compact

- insert Cleanius in bedpan washer-disinfector

#### cleaning Cleanius Eco

- empty the lower pan into the toilet
- clean and disinfect Cleanius completely

### Cleanius Eco with plastic liner and with liquid absorber before use

- lay in the plastic liner, insert the liquid absorber
- assemble Cleanius
- bring Cleanius to the patient
- remove genitalcover
- remove the lid

- place the patient on Cleanius
- place the genitalcover \*

#### after use

- remove the genital cover
- remove Cleanius from the patient
- clean the patients buttocks
- insert the lid
- insert the genital cover

#### cleaning

- remove the plastic liner, gelatinized urine and excrements can be disposed like a nappy into the dust bin. Important, do not dispose the plastic liner into the toilet!
- clean and disinfect Cleanius completely

\* The genital cover is a extra comfort part for more cleanliness. If it is uncomfortable with it, you don't have to use it.